


Folding the TQ for placement in the case



1° STEP

Hold the tourniquet in front of you by the base with the strap freely draping underneath and the D ring to the right.

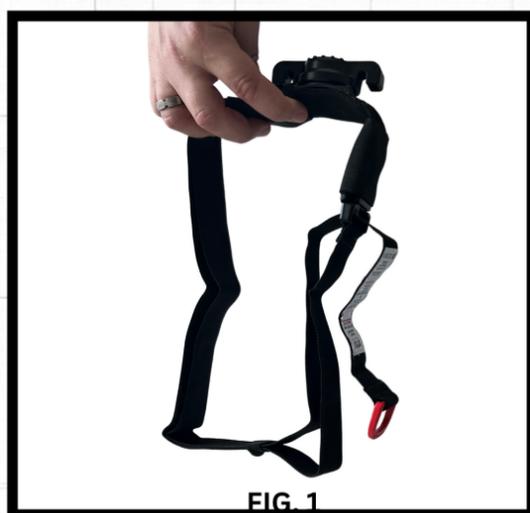

2° STEP

Pull the red D ring until the label is completely showing outside of the redirect loop




Then, grasp the strap so it cannot slide through the redirect loop, as shown in the 3rd picture, and position the elastic retention strap at the bottom of the loop, as shown on the right.




3° STEP

The tourniquet should now look like picture.

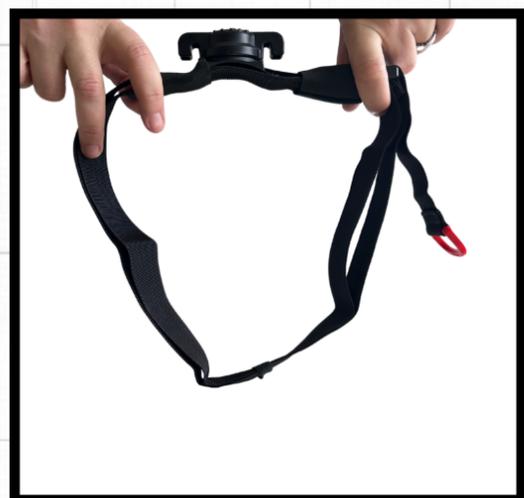


4° STEP

Now, continue to hold the tourniquet by the base and fold the end with the quick-release buckle under the tourniquet base, as shown in the picture.

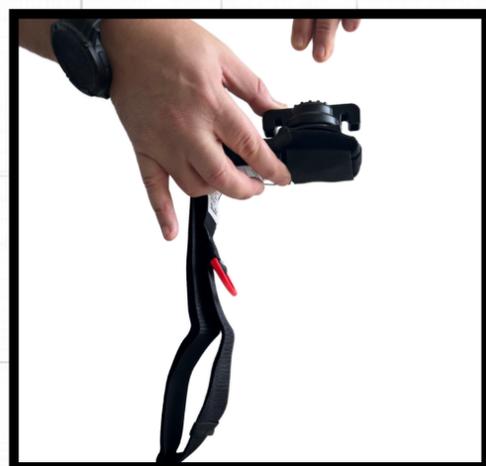

5° STEP

Grasp the red D ring and fold it three times onto the strap, as shown in the picture.

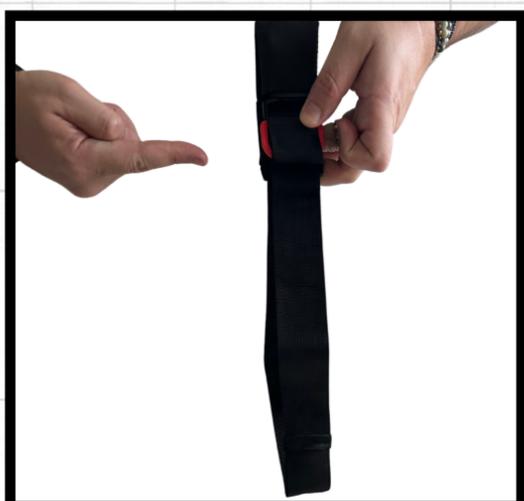

Folding the TQ for placement in the case



6° STEP

Then grasp the remaining loops of webbing and fold them up and under the tourniquet in equal links so that the folded tourniquet appears as in picture.



7° STEP

With the tourniquet folded in this manner, it can be inserted into the case with the long end that contains the red D ring going into the case first and the dial being pushed down until it seats in the space designed for it in the front of the case.



8° STEP

The tourniquet is now ready to place in case..



9° STEP

Now, when the tourniquet is needed, it can be removed by lifting directly out of the case, and it will naturally open into a loop ready to apply.

